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Please note the following cleaning instructions for the All-in-One drills:

Due to the surface texture of the drills, they should only be cleaned manually and in an ultrasonic bath. Place the products in an ultrasonic bath at a frequency of 40 kHz and a temperature of 45 °C for at least 5 minutes, enriched with (almost) pH-neutral, enzymatic cleaning agent. Deionized water (VE-water) should be used.

All products should be covered by the cleaning solution. The temperature of the cleaning liquid must not exceed 45 °C. After that, rinse 3 times with flowing, deionized water. A fresh, unused cleaning solution should be used for each product. The previous steps should be repeated until no visible contamination remains. Afterward, thoroughly rinse each product with deionized water (approximately 1 minute).

Further details and a detailed guide for preparation, as well as further disinfection, can be found under “Preparation before manual and automatic cleaning/disinfection” and “Manual cleaning and disinfection” in our reprocessing instructions R1 (Revision 13, 20.10.2023); available at <https://ifu.medentis.com/>.

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